


QUICK REFERENCE GUIDE

Staminalift Transfer System

IMPORTANT

The Transfer System is only to be used by trained operators. Always refer to the Operators Manual.

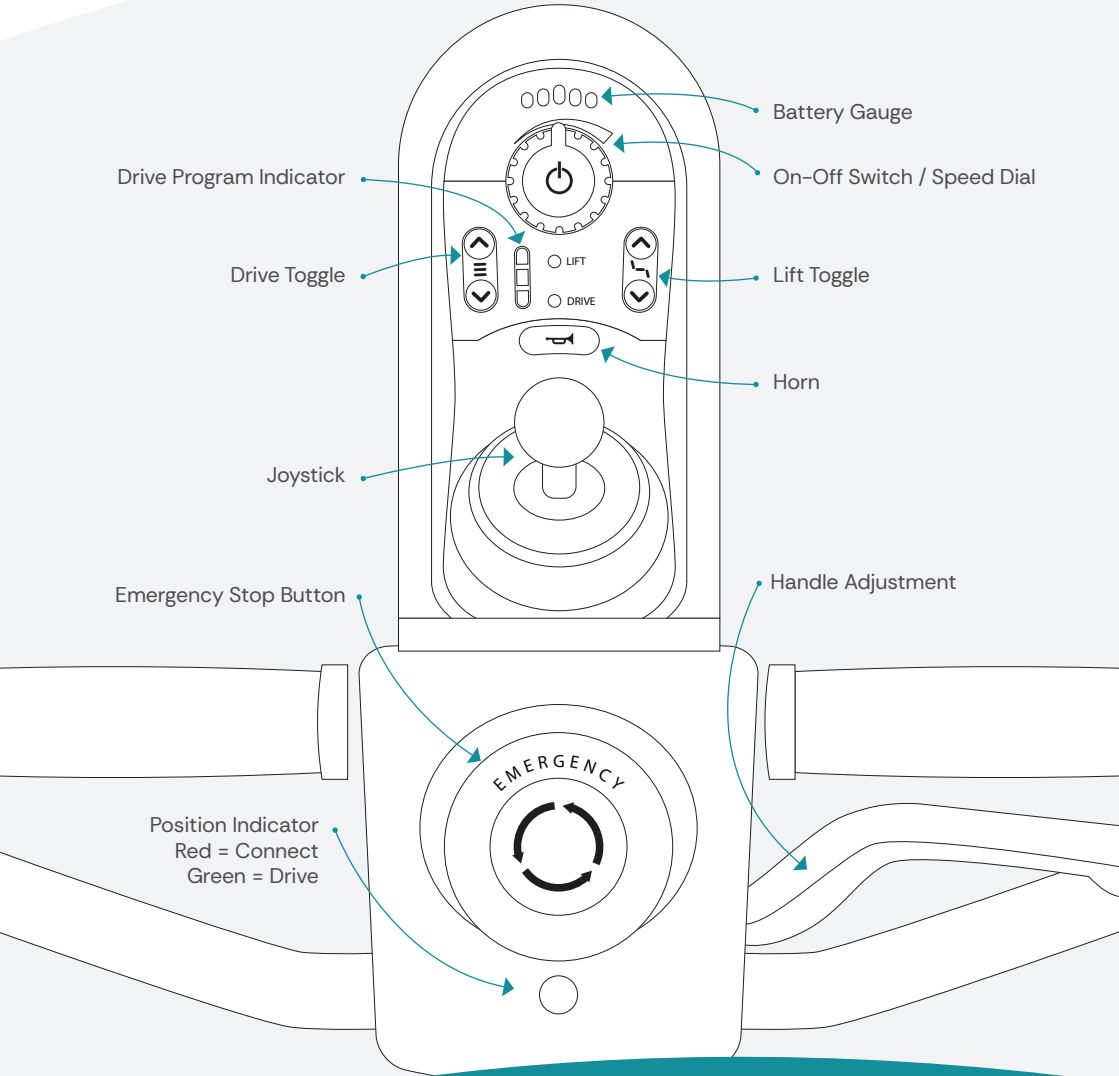

QUICK REFERENCE GUIDE

Staminalift Transfer System

IMPORTANT

The Transfer System is only to be used by trained operators. Always refer to the Operators Manual.



Ensure the Emergency Stop Button is released to operate.




Press to turn device on or off.



Controller is locked.
Enter correct code to unlock.



Power is on.
Batteries are fully charged.

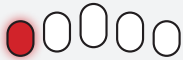

Batteries are low.
Charge immediately.



Drive Mode is selected.
Press to cycle between drive programs.



Lift Mode – use joystick to lift, lower, open and close the jaw.



Flashing red – there is a fault with the unit.
Report the fault immediately.



Ensure the Emergency Stop Button is depressed when not in use.

